

Congratulations to your new VoiSec. We have strived to deliver an easy-to-use quality product in three parts:

- the **VoiSec-button** for recording, storing and playing short speech messages.
- the soft rubber **holder** is used to prevent involuntary recording and to hold the VoiSec-button. The button should always be placed in the holder except when recording or changing batteries.
- the **attachment means** are used to attach the holder to other objects, e.g. the lid of a tin. VoiSec is delivered with adhesive magnets but other kinds of attachment means are available.

1. Playing a message



VoiSec should be placed in the holder when to play. Press the lid to play a recorded message. The **VoiSec Multimessage** will play all messages in a sequence from the start. Press again to stop playing.

2. Recording a new message




- Remove VoiSec from the holder, turn it over press the record button and keep it pressed. Do not block the little opening next to it.

- Wait for two beeps before talking. Talk straight towards the bottom surface of the button at a distance of approximately 10 - 20

cm.

- When the button is released, a two-beep signal and the recorded message should be played.
- The **VoiSec Singlemessage** will erase the previous message when recording a new one.
- The **VoiSec Multimessage** will add one message after another until the memory is full. After that, a new recording will erase all previous ones and start from the beginning again.

3. Changing batteries

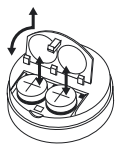


N.B.! Changing batteries will not erase any recorded messages.

- Open the battery cover at the bottom of the button by unscrewing fastening screw. Remove the cover.

- Insert two type SR44 or LR44 batteries with the flat end (+) facing outwards and the protruding end inwards. We recommend using batteries from Libego AB (available from your dealer).
- Re-mount the cover and re-fasten the screw well.
- Put the VoiSec-button back into the holder.

4. Attaching VoiSec to other objects

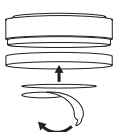

- Use the included adhesive magnet or other attachment means from the available accessories.

- Attach the magnet at the center underneath the holder.

- Make sure the surface to be stuck to is dry and clean.
- Remove any protecting material.
- Press the holder and the attachment means against the object to be stuck to.

5. Error handling

VoiSec is pressed but nothing is heard.

- No message is recorded.
- The button is not fully depressed.
- The batteries are not properly inserted.
- The battery cover is not fully closed.
- The batteries are worn or of the wrong type. When the batteries are beginning to run out of energy, VoiSec may have difficulties in instant re-playing. The button will manage about twenty more play-outs with some pausing in between.
- VoiSec has been shortcut by humidity. Remove the holder, the battery cover and the batteries. Let it dry and try again with fresh batteries.

The sound volume is too low.

- The message was recorded with too low a voice, the button facing the wrong way, beside or far away from the mouth.
- The hearing distance is too far.

The national warranties apply, contact your dealer if you suspect a manufacturing error.

6. Technical facts

Measures: diameter 39 mm, height w. tray 18.5 mm.

Weight: 26.8 g (0.94 oz)

Capacity: about 64 seconds recording.

Runtime: typically 200 playouts of a 30 second message (volume and voice pitch dependant).

Power: two 1.5 V type SR44 button cell batteries.

VoiSec may be cleaned with a dampened cloth but not immersed in water.

This product has been CE-certified by an authorised laboratory.

7. Disposal

Used batteries should be disposed of in accordance to local and EU regulations if applicable.

VoiSec should be handled as electronic waste. The plastics may be recycled.

8. Accessories

Batteries, holders and different products for fastening or wearing VoiSec may be ordered. Contact your dealer or see www.voisec.se for up-to-date information.

9. Consumer contact

Please contact your point-of-purchase or

Libego AB

**Diskettgatan 11B,
SE-583 35 Linköping,**

Sweden

E-mail: info@libego.se

Web: www.voisec.se

Phone: +46(0)13 342 01 60

Fax: +46(0)13 342 01 61

VoiSec is patented, a registered trade mark and the design is protected.