

6. Disposal

Used batteries should be disposed of in accordance to local regulations. VoiSec should be handled as electronic waste. The plastics may be recycled.

7. Accessories

Batteries, holders and different products for fastening or wearing VoiSec may be ordered. Contact your dealer or see www.voisec.se for up-to-date information.

8. Technical facts

Measures: diameter 42 mm, height with tray 18 mm.

Capacity: more than one minute.

Runtime: typically 400 playouts of a 30 second message (volume dependant).

Power: two 1.5 V type SR44 buttoncell batteries.

VoiSec may be cleaned with a dampened cloth but not immersed in water.

This product has been CE-certified by an authorised laboratory.

9. Consumer contact

Please contact your point-of-purchase or

Libego AB

Teknikringen 7

SE-583 30 Linköping

Sweden

E-mail: info@libego.se

Web: www.voisec.se

Phone: +46(0)13 342 01 60





Fax: +46(0)13 342 01 61

VoiSec is patented, a registered trade mark and the design is protected.

VoiSec User Manual

This information is also found at
www.voisec.se

VoiSec

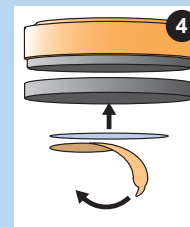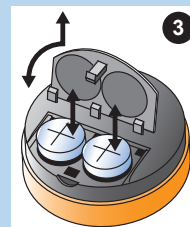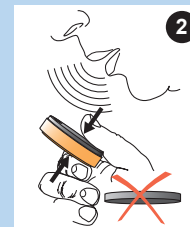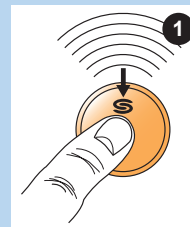

1. Playing a message

VoiSec should be placed in the holder when to play. Press the lid to play a recorded message. The **VoiSec Multimessage** will play all messages in a sequence from the start.

Press again to stop playing.

2. Recording a new message .

- Remove VoiSec from the holder, turn it over and keep it pressed together. Do not block the little protruding spike or the little opening next to it.
- Wait for two beeps before talking.
Talk straight towards the bottom surface of the button at a distance of approximately 10 - 20 cm.
- When the button is released, a two-beep signal and the recorded message should be played.
- The **VoiSec Singlemessage** will erase the previous message when recording a new one.
- The **VoiSec Multimessage** will add one message after another until the memory is full. After that, a new recording will erase all previous ones and start from the beginning again.

3. Changing batteries

N.B.! **Changing batteries will not erase** any recorded messages.

- Open the battery cover at the bottom of the button by pressing down at the notch on the outside. Remove the cover.
- Insert two type SR44 batteries with the flat end facing outwards and the protruding end inwards.
- Re-mount the cover and close well.

4. Attaching VoiSec to other objects

- Use the included adhesive patches or other attachment means from the accessories assortment.
- Attach the patch at the centre under the tray.
- Make sure the surface to be attached to is dry and clean.
- Remove any protecting material.
- Press the tray and the attachment means against the object to be attached to.

5. Error handling

VoiSec is depressed but nothing is heard.

- No message is recorded.
- The button is not fully depressed.
- The batteries are not properly inserted.
- The battery cover is not fully closed.
- The batteries are finished or the wrong type.
- The little protruding spike underneath is not fully depressed by the attachment tray.
- VoiSec has been shortcut by humidity. Remove the holder, the battery cover and the batteries. Let it dry and try again with fresh batteries.

The sound volume is too low.

- The message was recorded with too low a voice, the button facing the wrong way, beside or far away from the mouth.
- The hearing distance is too far.

The national warranties apply, contact your dealer if you suspect a manufacturing error.